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TEXPEL MICRO ANTI-MICROBIAL FINISH
 Texpel Micro is a unique patented technology which inhibits the spread and growth of harmful microbes such as bacteria, fungi, spores, and viruses. This permanent silver ion treatment has been independently tested and proven to provide 99.9% of antimicrobial properties even after 50 washes.

Face Mask Handling:

- Wash your hands before handling or putting on the mask
- Hold the elastic at the side of the mask and place over the face (covering mouth and nose).
- Pass the elastic around your ears and use the adjusters to get a secure fit. Where applicable, pinch the nose bridge to adjust it over the nose.
- Remove the mask by the ear loops and put into your washing machine.
- Wash your hands after removal of the mask.

1 Wash your hands before handling or putting on the mask.

2 Hold the elastic at the side of the mask and place over the ace (covering mouth and nose).

3 Pass the elastic around your ears and use the adjusters to get a secure fit.

4 Remove the mask by the ear loops and put into your washing machine.

5 Wash your hands after removal of the mask.

MANUFACTURER
 Portwest, IDA Business Park, Westport, Co. Mayo, F28 FY88, Ireland

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